



HULL UNIVERSITY HOSPITAL VP7000

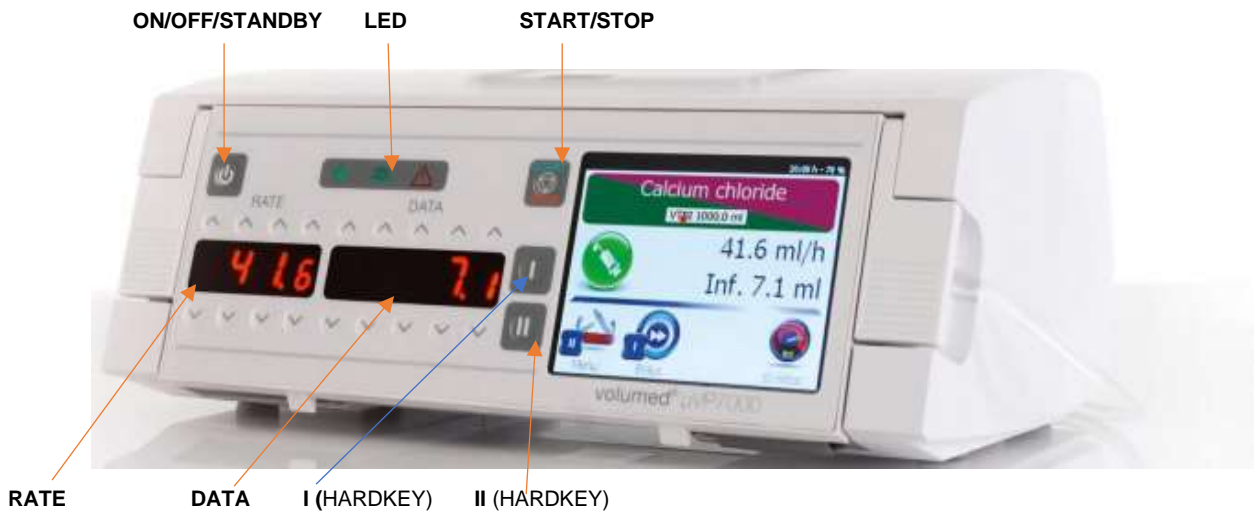

LOADING THE GIVING SET

- Open pump door by lifting latches upwards. Door opens toward you.
- Place line vertically within body of pump with Robson clamp closed.
- Close pump door
- Pump performs Stop Flow Test, confirming Arcomed giving set, correctly inserted.

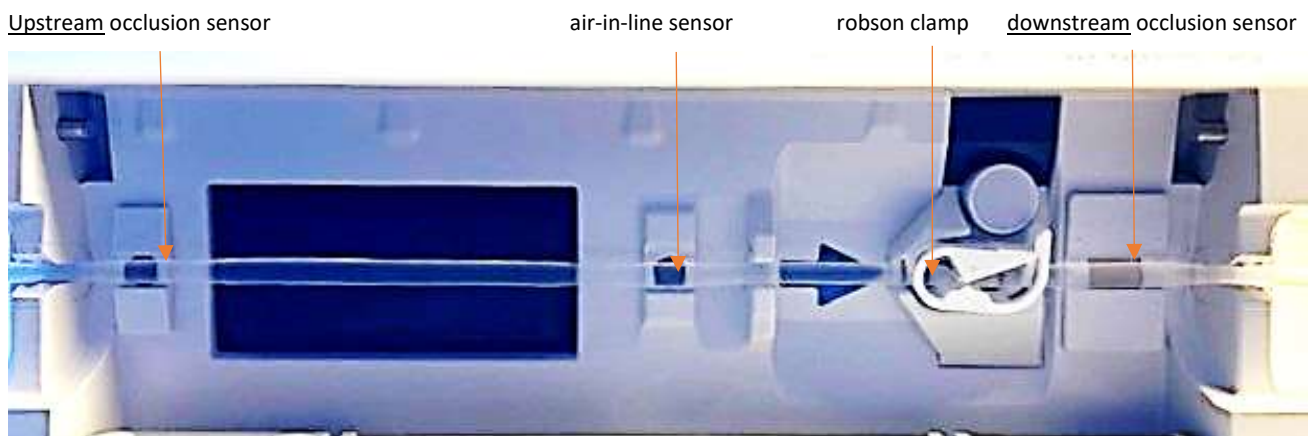

HULL UNIVERSITY HOSPITAL VP7000

PROGRAMMING/STARTING INFUSION

Select **ml/hr** or **V/T** by pressing on Touchscreen.

ml/hr

- Enter **RATE** in Rate Window
- Enter **VTBI** in Data Window
- Press **START**

V/T

- Enter **VTBI** in Data Window and confirm.
- Enter **Time** to be infused in Data Window and confirm.
- Press **START**

**ALWAYS FOLLOW INSTRUCTION ON
TOUCHSCREEN**

- **RATE** window shows ml/hr when pump is infusing.
- **DATA** window will display cumulative dose administered when pump is infusing.
- **TIME REMAINING COUNTS DOWN** to end of infusion

HULL UNIVERSITY HOSPITAL VP7000

- **INFUSION HISTORY/TIME REMAINING**

- Press **Blue Drop** icon in shortcut on touchscreen.
- Time remaining counts down (below icon)
- 1st page displays rate, VTBI, infused in ml, cumulative infused in ml, volume, and time remaining (dynamic) since infusion commenced.
- Press key **II** for 2nd page, displays date, time, event/log history (press **down arrow** for further pages)
- See also in Menu (**i status/help**).
- Records/retains previous 1500 events.

- **PRESSURE READING**

- Line pressure: displays default pressure. Current pressure shown below pressure icon in shortcut on touchscreen.
- Can be raised/lowered according to clinical need/hard limits, will return to default when pump turned off/on.

HULL UNIVERSITY HOSPITAL VP7000

- **ZERO VOLUME**
 - Press **STOP** (stop infusion)
 - Press **Menu**
 - Press **II** (hard key)
 - Zeroes cumulative volume.
-
- **ALERTS & ALARMS**
 - **Alert:** pump is usually still infusing and is advisory eg power fail
 - **Alarm:** the pump has stopped infusing eg occlusion, infusion near end, bag empty
 - To **silence** auditory alarm press touchscreen or **II**
 - Press **START** when problem addressed.
 - **Constant Alarm:** internal error. Press/hold **ON/OFF** button until silences alarm. *Do not use on patient, return to EBME.*
-
- **STANDBY MODE**
 - Briefly press the **ON/OFF** key once (infusion must be stopped)
 - Pump will not alarm.
 - Pump retains settings entered.
 - To cancel standby, press **touchscreen** or briefly press **ON/OFF** key.

HULL UNIVERSITY HOSPITAL VP7000

BOLUS/PRIME

- To prime line via pump (*patient must be disconnected*). PP giving sets cannot be auto primed due to AirLock.
- Bolus Function.